


Update the iMRS prime firmware 2.xx with 2.00 already installed

1. Download the file (if you get a zip file, unzip it) and copy the .ppkg file on an USB-stick, which needs to be new or at least empty.

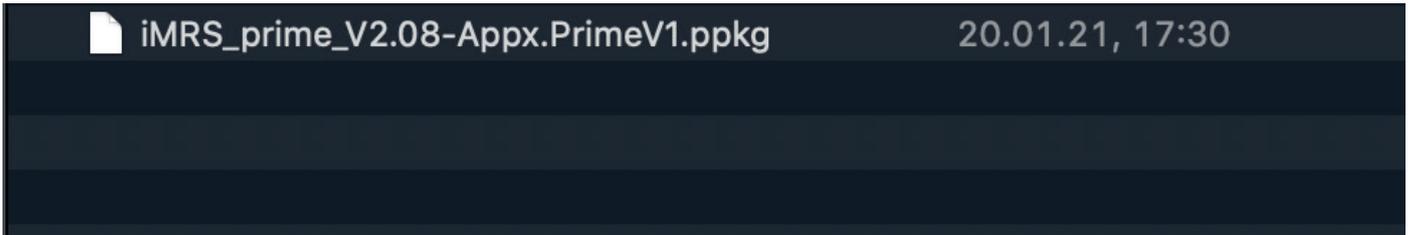




2. Start the iMRS prime control unit and click „Start“ after booting.

3. Plug the USB stick into one of the USB sockets on the left side of your iMRS Prime control unit.




4. Under Settings (wheel symbol in the main menu), select „System information“ and then select „System update“.

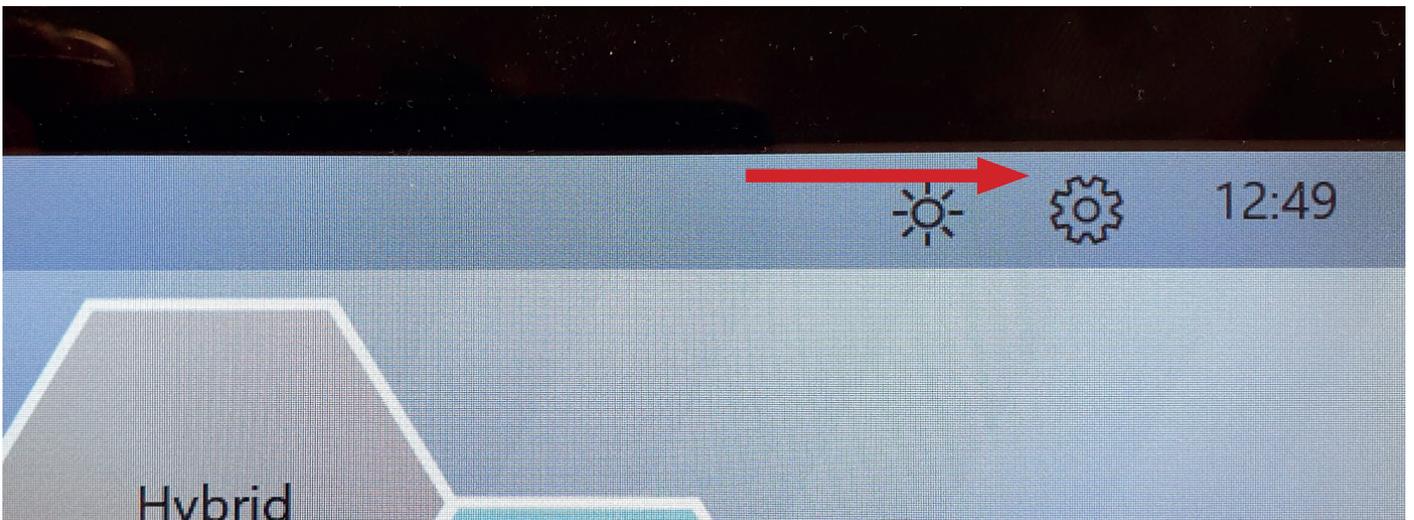

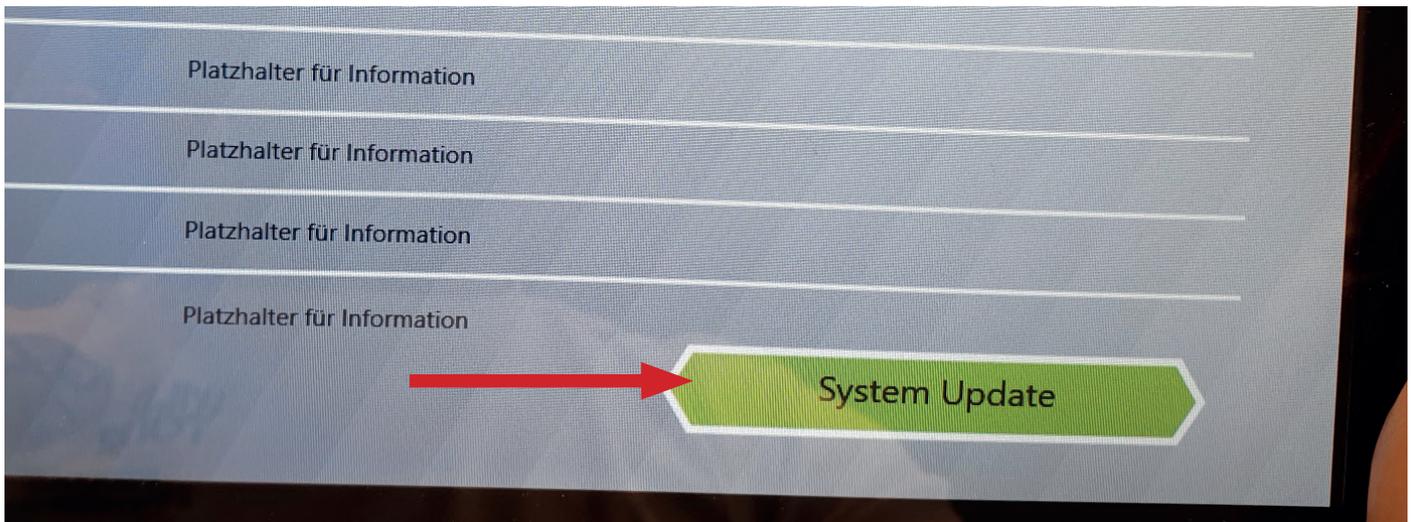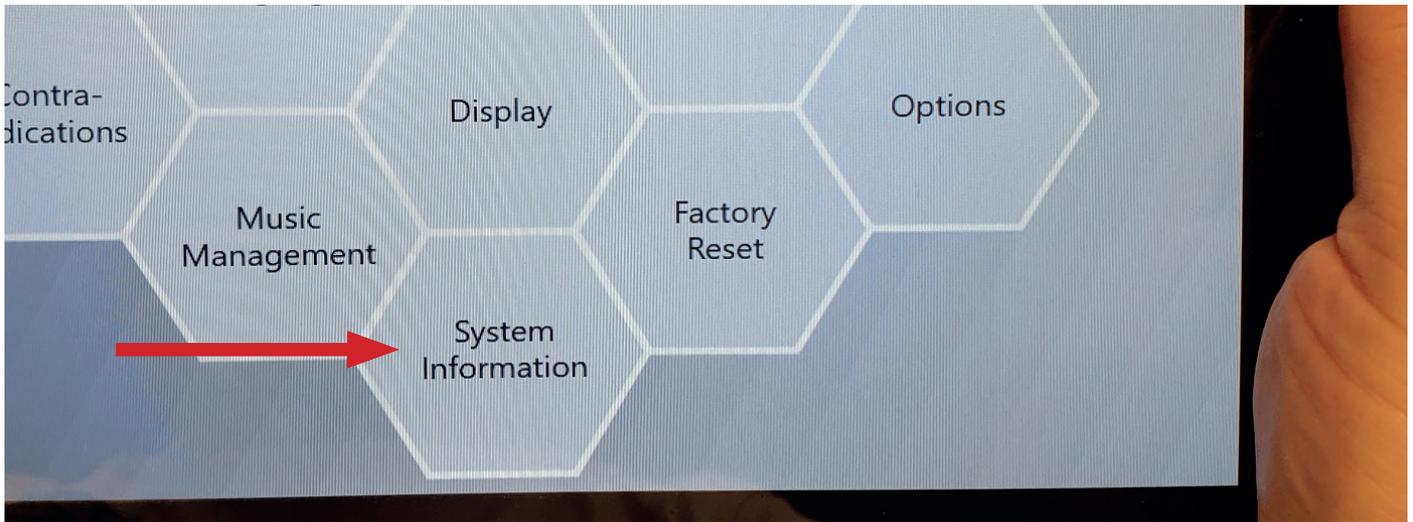

5. Tap „Search for Update“. It should now state that the update has been found . Tap the „Start Update“ button.

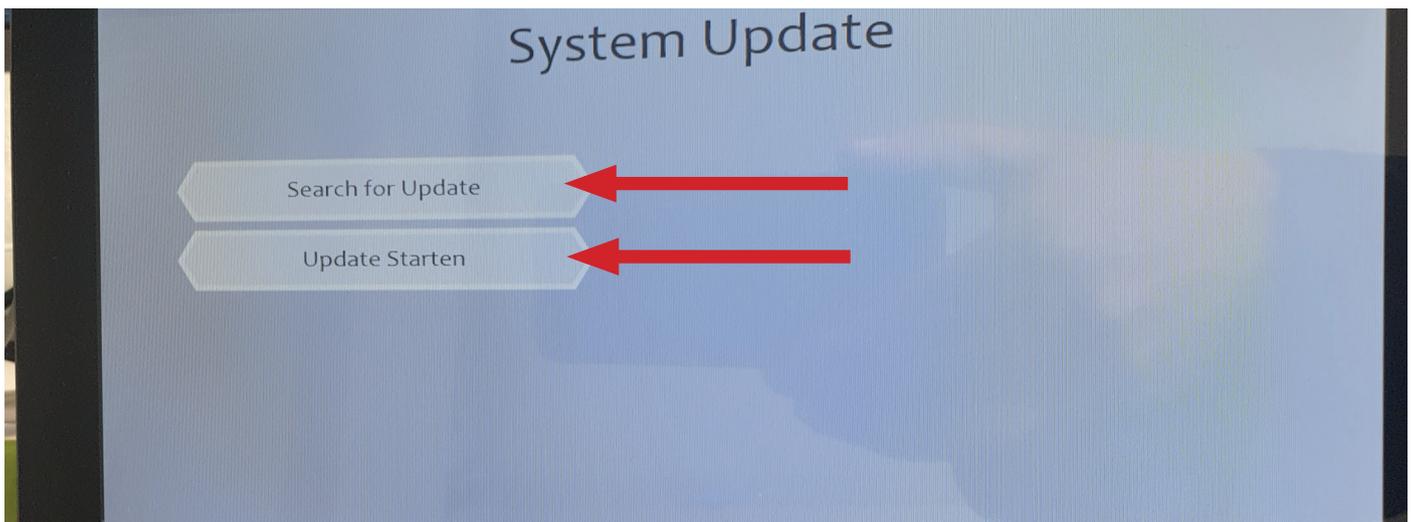

IMPORTANT NOTE: Wait for about 4 minutes until the System is updated (it reboots two times). Do not switch it off or press any buttons until the update is complete.

6. After any firmware update, you need to re-unlock the level of system that you've paid for. To do this:

- First locate your Activation Key code (most shipments have this recorded on the invoice you received or on a sheet of instructions that came with the system; if you don't have this code, please contact your Swiss Bionic Solutions country office).
- Press the Settings (gear) icon at the top right corner of the screen.
- Press the Options field on the right edge of the Settings display.
- A display will appear at the top of the page, showing „The following option is activated: Basic“ (even though the system level you paid for may be above this).
- Plus two boxes at the bottom of the page: „Activation key“ and „Unlock“.
- Press the Activation Key box and a keyboard should appear on screen. Assuming you have it, please enter your Activation key code (in numbers and upper case letters; be careful of the difference between a capital O and a 0 - zero).
- Press Unlock.
- The top of the Options page should now be showing the level of system that you purchased: Basic, Advanced, Expert, Hybrid or Trial, which means all the functions of your system level are now activated.
- Return to the Home screen.

7. You may also see a new pop-up window appearing, titled Audio Configuration, and a melody will play for you to confirm that you can hear it from the unit's internal speakers. This only needs to be confirmed once, or after factory settings, a reboot or forced reboot.