


Update the iMRS prime firmware

1. Unzip and copy the 4 .wim files on an empty USB stick

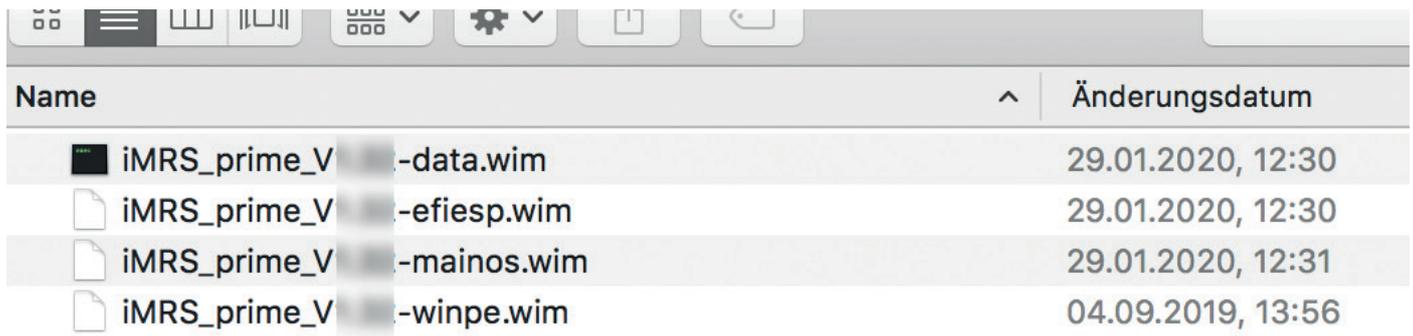


Name	Änderungsdatum
iMRS_prime_V -data.wim	29.01.2020, 12:30
iMRS_prime_V -efiesp.wim	29.01.2020, 12:30
iMRS_prime_V -mainos.wim	29.01.2020, 12:31
iMRS_prime_V -winpe.wim	04.09.2019, 13:56



2. Start the iMRS prime control unit and click „Start“ after booting.

3. Insert the USB stick into one of the USB sockets of the iMRS prime control unit.




4. Under Settings (wheel symbol in the main menu), select „System information“ and then select „System update“.

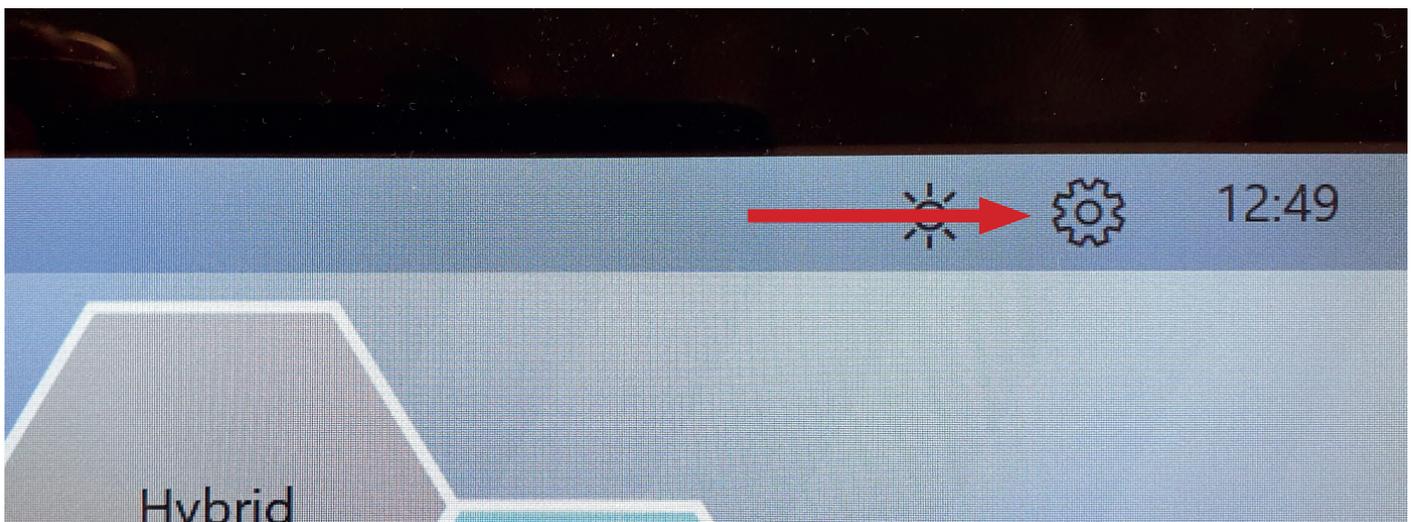

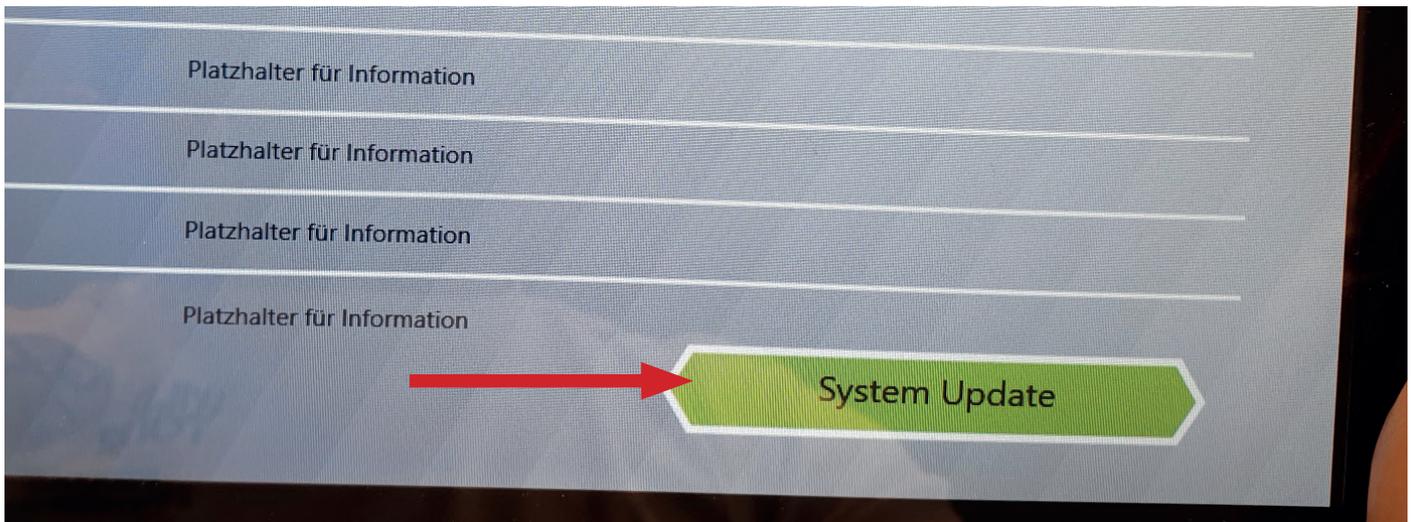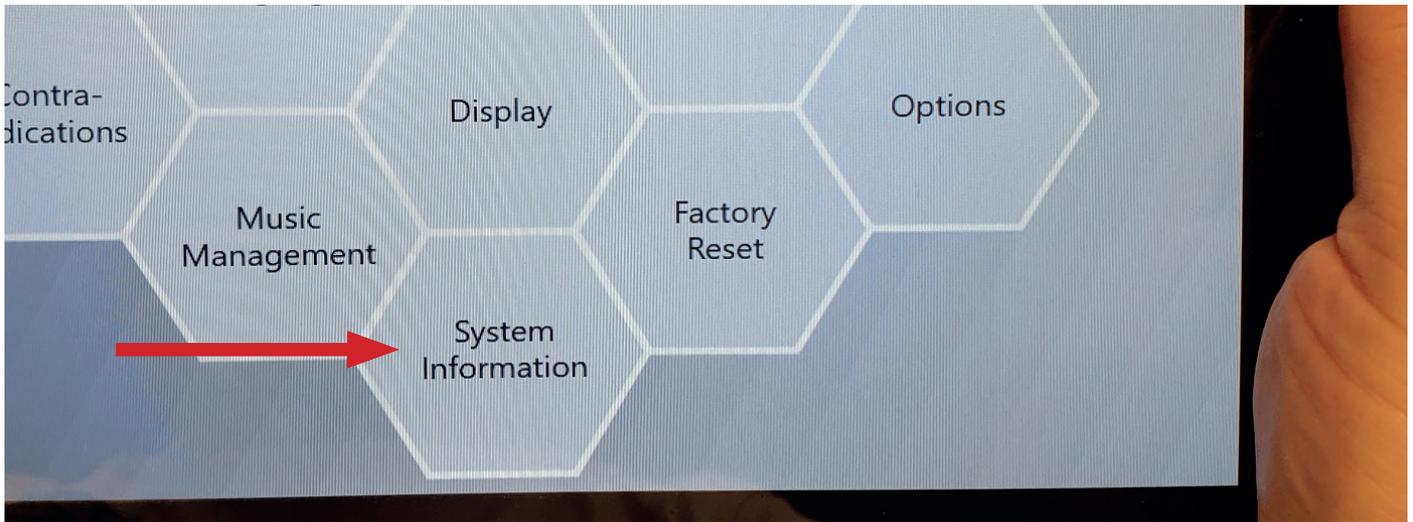

5. Tap „Search for Update“. If the system can find the update files, tap the „Start Update“ button.

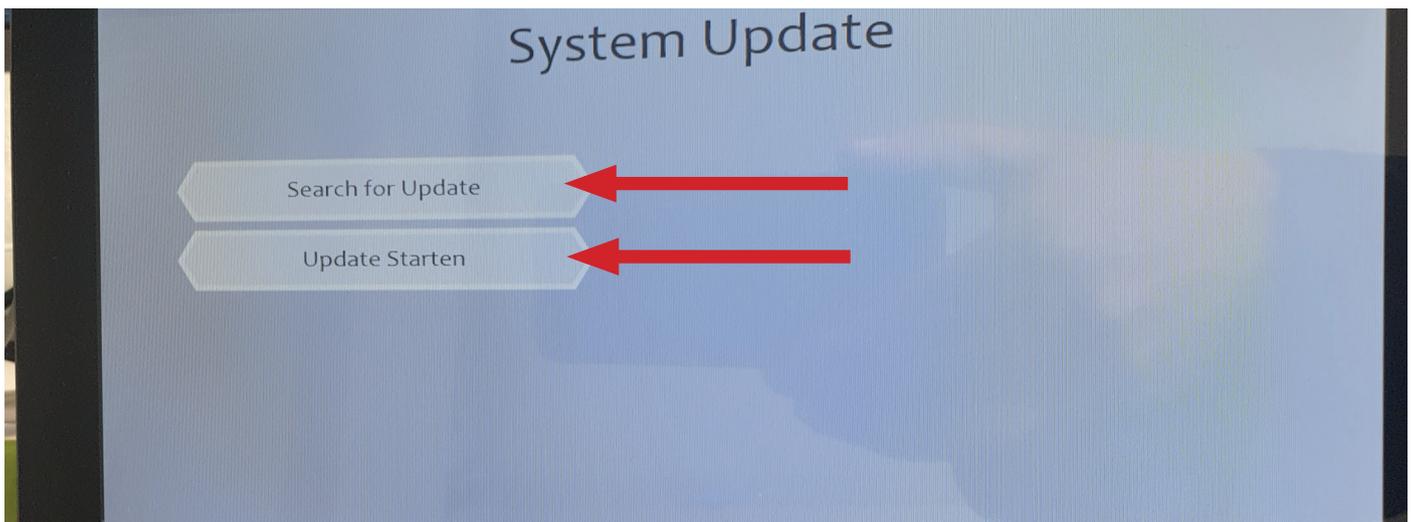

IMPORTANT NOTE: During the update process the iMRS prime display will change multiple times while extracting and uploading the necessary files. Please do not touch the device/ display and do not interrupt the process until the display will show you again the start screen (with the contraindications). The process will take at least 10-15 Minutes!